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INSTRUCTIONS to fill out the daily log

The daily log is an important tool for monitoring the progress of the treatment. We recommend that you fill out this log on a daily basis and transfer the data to our online follow-up software (via our website www.sos-bdwetting.com) in order to benefit from our computer-generated personalized graphs. If you do not have access to internet, you can fax the log after every 10 to 15 nights of treatment.

Complete the following information:

• **DATE and BEDTIME**

• **OZ / mL**: liquid intake 10 minutes before going to bed

• **TIME OF URINATION** ((if the child urinates more than once per night, write them all on the same line – one night per line) :

1. **IN THE SENSOR UNDERWEAR (indicate the size):**

LARGE: the sheet is very wet and has to be changed (or the diaper is full)

MEDIUM: the stain on the underwear measures more than 3 inches (7.5 cm) (or the diaper is mildly to moderately wet)

SMALL: the stain on the underwear measures less than 3 inches (7.5 cm) (or the diaper is barely wet)

A/P: indicate who woke up the child by "A" or "P":

• **A for "alarm"** : if the child woke up on his own because of the alarm

• **P for "parent"**: if a parent had to wake the child up following the triggering of the alarm because he was still sleeping

2. **IN THE TOILET WITHOUT ALARM**: the child woke up by himself (without the alarm and without anyone's help) because he felt the urge to urinate and went to the bathroom

• **NO URINATION**: put a "X" in this box if the child did not urinate in the sensor underwear or did go to the bathroom.

If the time of the urination is not known

Specify the size of urination by putting a "X" in the appropriate box (Large, Medium, Small or Toilet urination without alarm); this will allow us to know the number of urinations per night.

If the device is not used for one or more nights because of sickness, sleepover, visitors, vacation or other reasons

Do not fill out the daily log for these nights. Instead, write the date of the last night the device was used, and on the following line, write the date of the next night it was used. For those nights, limit fluid intake one hour before bedtime.

End of treatment: the treatment should be continued until the child has completed 14 consecutive nights without wetting.